



CellGenius



**CellActivator**

User manual

"Do you want to feel energized and in a good mood every day?

**CellActivator** makes it possible - completely natural and without any side effects!"

## **"What can CellActivator do for you?"**

This ingenious device provides your body with a real energy boost - and all in a completely natural way without much effort! With **CellActivator**, you supply your body with extra oxygen/energy, so you'll feel better all around.

## **How does it work?**

**CellActivator** generates what are known as Light-Air Ions, which improve your indoor air quality in a way similar to what you find in the forest or by the sea. Studies show that more oxygen and less stress lead to a healthier heart and brain. And the best part: It enhances your overall sense of well-being!"

## "Why is oxygen so important?"

We spend a lot of time in poorly ventilated spaces, and that's not just uncomfortable but also bad for our health. On the other hand, good, energizing air has many benefits:

- Improved fitness and youthfulness
- Better health
- Faster recovery after exercise
- Stress reduction
- And much more!

Properly breathing energizing air supports and enhances every physical and mental aspect.

With **CellActivator**, you can easily achieve all of this."

## Application and Science

Whether you're working, relaxing, or sleeping, the **CellActivator** is always there for you! Simply let the device run day and night and experience the sensation of a 'mini air spa,' as if you were on a wellness vacation.

### Scientific Facts:

- Up to **90%** fewer viruses and germs
- Increases cell regeneration by **22%**
- Enhances the immune system by **20%**
- No side effects!

### Expert Recommendations:

- *Prof. Dr. Peter C. Dartsch:*

"Highly recommended for improved well-being!"

- *Dr. Ulrich Schmelz:*

"A true health booster!"

# Where and How CellActivator Enhances Your Quality of Life

## **Boost Your Immune System with CellGenius**

Especially in these times, a robust immune system is invaluable. **CellActivator** helps provide your body with the extra protection it deserves and needs. Here's something you can directly influence to stay healthy!

## **Breathe Better, Live Better with CellGenius**

Short on time for a walk in the healthy outdoors?

**CellActivator** transforms stuffy, stale indoor air into invigorating, clean air. It helps you breathe easier, making you feel more comfortable!

## **Say Goodbye to Stress with CellGenius**

Stress is annoying and can even make you sick. It contributes to 80% of all illnesses and shortens life. A few deep breaths of energizing air, and you'll feel much more relaxed and balanced. **CellActivator** makes it possible!

## **CellGenius for Better Sleep**

Can you remember the last time you slept really well?

**CellActivator** provides energizing air that helps your body recover effectively during sleep. You'll wake up more refreshed - every day! Discover how **CellActivator** enhances your well-being and adds that extra touch to your life!"

# Lieferumfang



**CellActivator**



cleansing brush



wrist strap



USB-C charging cable




power adapter



opening slider


# Funktionen

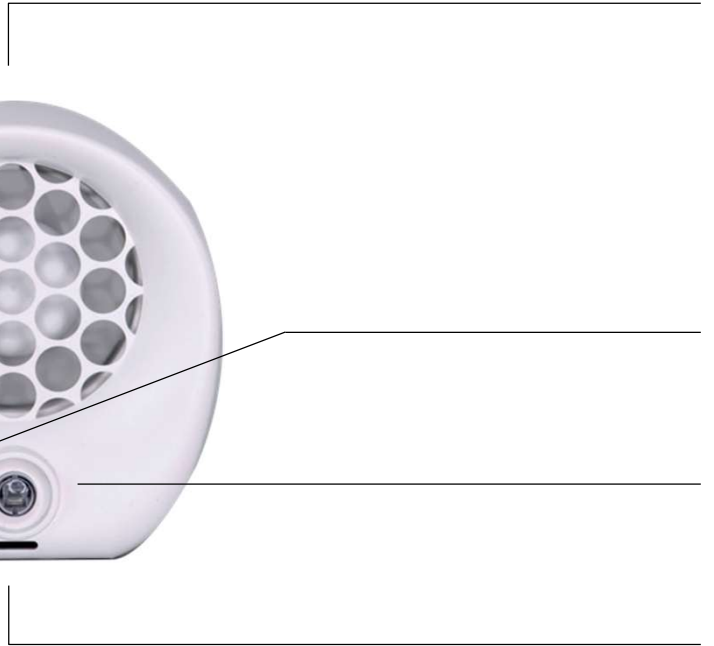






air intake

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

USB-C port



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LED on/off switch




slot for wrist strap



Luftauslass


**Please charge the CellActivator before first use. It may take up to 4.5 hours for the device to fully charge.\***

### **To turn on the device –**

Press the On/Off button: 


- Brief illumination
- Fan for air activation is turned on

### **To activate the Anti-Stress button**

Continue to press On/Off button: 

- Brief illumination accompanied by a short beep
- Every 20 minutes, there will be three short beeps as a reminder for deep breathing and stress relief

### **To turn off the device –**

Keep pressing the On/Off button:   
(The fan will stop)

### **When the battery is low:**

- A low battery level is indicated by the LED flashing briefly at regular intervals

\* The **CellActivator** is charged using the included USB-C cable with a power adapter or a USB port. The battery life is up to 30 hours. The device is designed to be operated continuously (24 hours) when connected to the power adapter."



**LED diode is blinking continuously.**

What could be the reason?

For more information, please visit our website

**[www.cellgenius.at/faq](http://www.cellgenius.at/faq)**

# Cleaning



Make sure that the **CellActivator** is not connected to the power. Press the On/Off button to turn off the **CellActivator**



Do not use cleaning agents or polishes to clean your **CellActivator**. Wipe off dust from the surface with a dry or lightly dampened cloth.



Im Dust may accumulate inside your **CellActivator**. If this happens, open the casing carefully. Insert the opening slider into the groove at the bottom and move it with gentle pressure towards the locking mechanism. Afterward, you can gently pull apart the two casing parts."



Use the included brush for cleaning to remove all dust deposits. Please be extremely careful when removing dust and deposits from the red wire to ensure the functionality of the **CellActivator** is maintained and consistent performance is guaranteed."



CAUTION UNPLUG



Carefully reassemble the casing parts along the guide rail system. Ensure that the locking mechanism at the top of the **CellActivator** snaps in first. A slight slant of the casing parts can be advantageous. Then, connect the **CellActivator** with the USB-C cable and turn it on to verify its functionality.

---

## IMPORTANT NOTE

- Only perform the cleaning procedures outlined in this user manual
- Before cleaning, ensure that the **CellActivator** is disconnected from the power source and turned off. If the **CellActivator** is not functioning, first verify that the power outlet is supplying electricity and that the USB-C cable is properly connected.
- Use and store the device only in dry indoor environments at room temperatures.
- Please clean your **CellActivator** with the provided cleaning brush in the case of heavy dust buildup or high levels of dust accumulation

# Safety Notice

**PLEASE CAREFULLY READ THE "SAFETY NOTICE" SECTION IN THIS CELLGENIUS USER MANUAL BEFORE USING THE CELLACTIVATOR.**

## **CHARGING AND STORAGE**

Please charge the **CellActivator** before initial use to ensure the battery is fully charged. Do not store the **CellActivator** in high-temperature or humid environments (operating temperature: 0°C ~ 60°C).

## **BATTERY SAFETY PRECAUTIONS**

If you need to replace the battery (once its lifespan is reached), please contact CellGenius customer service.

**CAUTION:** Improper handling of the battery installed in this **CellActivator** can cause fire and burns. Do not disassemble, short-circuit, heat above 60°C, or burn the battery (risk of explosion). Keep the battery out of reach of children.

## **SAFETY AND HANDLING**

- The **CellActivator** is not a toy and should not be accessible to children.
- Do not cover the **CellActivator** while it is in operation.



## **DISPOSAL NOTICE**

- CellGenius products are made from high-quality, reusable materials. Please dispose of these products responsibly and recycle them if possible.
- You must remove the battery before disposing of the device.
- Dispose of or recycle the battery in accordance with local regulations.
- This symbol indicates that in the EU, you should not dispose of this product in regular household waste. Please ensure proper disposal to avoid potential environmental or health hazards from improper waste disposal and to support material recycling. Use the available return and collection systems or contact CellGenius if you wish to return your old device for eco-friendly recycling.

## 2-YEAR-WARRANTY

### **GENERAL TERMS OF THE 2-YEAR LIMITED CELLGENIUS WARRANTY**

- If within 2 years from the date of purchase or delivery of your **CellActivator** defects occur due to faulty material, workmanship, or function, we will repair or replace it after a professional inspection. If a part is no longer available or out of production, we will provide a functional replacement part.
- If you purchased your **CellActivator** within the EU, this warranty only applies if (1) the **CellActivator** is in the country where you purchased it, or (2) if you are using the **CellActivator** in Germany or Austria.



# Warranty

- The warranty begins on your date of purchase or the delivery date, whichever is later.
- Repairs will only be carried out if you can provide proof of purchase. Without a proof of purchase, you will be responsible for all costs.
- All work will be performed by CellGenius or its authorized partners.
- Any parts replaced become the property of CellGenius
- The warranty period is not extended by repairs or exchanges.

If your device requires service, please contact CellGenius customer service so we can discuss the best options. If your **CellActivator** is still under warranty and the repairs are covered by this warranty, it will be repaired free of charge.

# Warranty

## **WHAT DOES THE WARRANTY NOT COVER?**

CellGenius is not responsible for any costs associated with the repair or replacement of your product due to:

- Damage from improper use, abuse, or acts of force, such as dropping it on the ground.
- Damage due to non-compliance with the recommended instructions for device maintenance.
- Damage caused by external factors, such as transport, power outages, or power surges.
- Repairs or modifications not performed by CellGenius or its authorized partners.
- Use of parts or accessories that are not from CellGenius.
- Normal wear and tear.

Falls du Fragen zu dem, was CellGenius bietet oder zur Garantie hast, kontaktiere bitte den CellGenius-Kundendienst. Du findest alle Kontaktinformationen auf Seite 23.

If you have any questions about your CellActivator, please contact CellGenius customer service. Alternatively, you can also reach us through our website **[www.cellgenius.at](http://www.cellgenius.at)** If your device needs repair, please call CellGenius customer service so that we can inform you about your options. If your CellGenius device is still under warranty and the repair falls within the warranty coverage, the repair is free of charge.

### **IMPORTANT PRIVACY INFORMATION**

If you provide us with information about a third party, you confirm that this person has consented to the disclosure and processing of all their personal data to us and is well informed about the purpose of using this data.

Upon request, you can obtain a copy of the information we have recorded about you for your review and potential correction. For quality assurance purposes, we may listen to and record communications if necessary.

**LOOK AFTER  
YOURSELF AND THE  
HEALTH OF YOUR  
LOVED ONES.**



**SCIENTIFICALLY  
SUPPORTED.**



IT'S LIKE HAVING THE  
RELAXING ATMOSPHERE OF  
MOUNTAIN OR SEASIDE AIR  
RIGHT IN YOUR HOME.



## CellGenius Kundendienst



**Internet**  
[www.cellgenius.at](http://www.cellgenius.at)

**E-Mail**  
[kundendienst@cellgenius.at](mailto:kundendienst@cellgenius.at)

[Kundendienst](#)

### **CellGenius CellActivator**

Rated Voltage: 5V DC

Rated Power: 0.4 W

Dimensions: L 118 x W 99 x H 103



CellGenius



[www.cellgenius.at](http://www.cellgenius.at)